


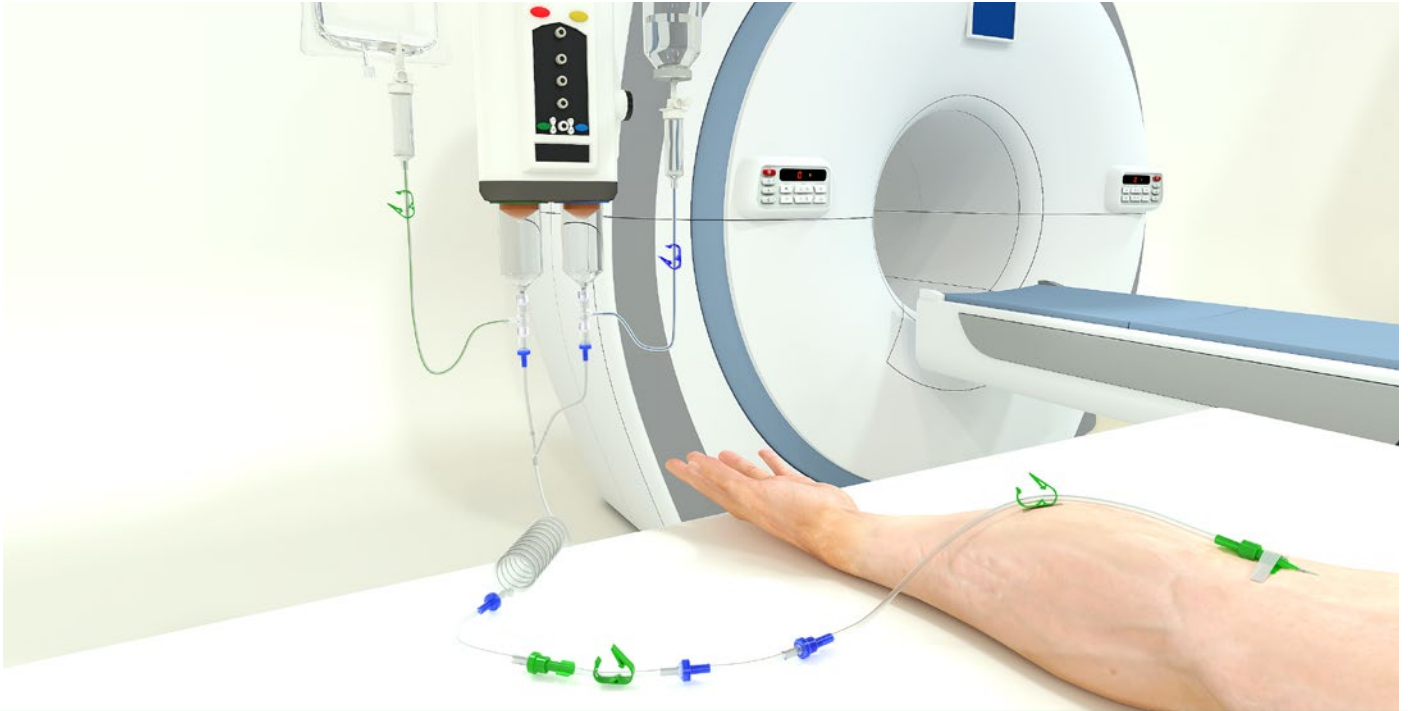

## **SAVE** SAFE

Save on cost, time, and waste without compromising on your patients' safety.



## SAVE

save on cost time and waste, thanks to our more efficient design

### SAVE COST



By using the most expensive part of your infusion system for up to 24 hours across multiple patients, you can reduce material costs by up to 90% every day.

### SAVE TIME



You no longer need to replace syringes and prime the filling lines between each patient. This allows you to focus on other tasks or achieve a shorter patient turnover.

### SAVE WASTE



The Transflux® system is designed to guarantee full patient safety while reducing materials. By reusing the majority of the contrast lines and syringes for 24 hours, you save on contrast media, syringes, and fill lines. The reduction in waste can be as much as 90% compared to single-use systems.

## Sustainable combination

The combination of Transflux® & Transset® provides an efficient, safe-to-use, and cost-effective solution for your contrast media deliveries. You no longer need to change the syringes on a 'per-patient' basis. The Transflux® protects the reusable filling set, Transset®, between patients. The Transset® can be used for 24 consecutive hours when used in combination with Transflux® and as prescribed in the instructions for use. Only the Transflux® has to be changed on a 'per-patient' basis. This makes the system extremely cost-effective, reduces waste, saves time, and consequently increases patient throughput.

# Transflux®

unique safety features



## SAFE

our product stands out with its unique safety features

### DISCONNECT FEATURE



Our innovative and patented disconnect feature enables the monitoring of blood reflux and prevent extravasation. Additionally, this feature provides continuous access to the vein, ensuring prompt intervention in the event of an unforeseen reaction, allowing immediate administration of drugs.

### TWO VALVES TWO SAFETY ZONES



The two one-way valves, separated by a fluid pressure interaction zone, prevent reflux from contaminated fluids, providing additional protection for the multi-use Transset®. When the examination is over, you can easily disconnect the patient from the contrast media set, leaving the double valve part on the Transset®, creating a seal against ambient contamination until the next patient arrives and can be connected to the Transset®.

## Safest combination

We all strive to be as efficient as possible, and who doesn't want to save on cost, time, and waste? However, when doing so, it's essential to ensure the safety of both your workers and patients. The Transflux® and Transset® are specifically designed around this principle!

Single-use products are typically employed with infection control in mind, but they add cost and waste to your procedures. With the Transflux® & Transset® combination, you get the best of both worlds: the largest and most expensive part is multi-use, while the smallest and least expensive part of the system is disposable, reducing both cost and waste. The single-use part, the Transflux®, shields the multi-use part, the Transset®, from cross-contamination risks. Additionally, the Transflux® acts as an environmental protection cap for the Transset® between patients, further reducing the risk of infections. Several studies demonstrate that the Transflux® & Transset® is safe to use, with no instances of cross-contamination or environmental contamination during 24-hour use.

# Transflux® & Transset®

the safest combination



## P&R MEDICAL BV

Wetenschappark 5  
3590 Diepenbeek, Belgium  
+32 (0)11 23 46 82  
info@trans-flux.com

[www.trans-flux.com](http://www.trans-flux.com)

follow us on social media

